# **ROOMBA**Indoor Rollator **C**€

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### Indication

Walking device – for indoor and outdoor use – for compensation and improvement respectively of restricted ability of walking.

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# Height adjustment handgrips

Take the C-cap out, draw the handle tube up and to the desire height. As Figure 3. Insert the C-cap though the hole.

NOTE: Refer to the INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.

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# **Opening/folding the rollator**

NOTE: Refer to the FOLDING/ASSEMBLY WARN-INGS in the SAFETY SUMMARY of this instruction sheet. The figures are without bag for best view.

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### **Opening**

Support the front oft the tray with one hand and lift the walker handle upwards, pull the tray tot he direction as Figure 1. The tray is completely open with a clear "click".

To ensure it is fully open, support the handle with one hand and press on the tray another hand to the direction of Figure 1. The rollator is completely opened for use with a clear "click".

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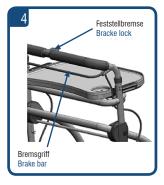
### Foldina

Grasp the tray folding bar and lift to the direction as Figure 2 to release the tray and fold down.

Grasp the handle with one hand and front wheel bar center with another, drag the rollator folding bar with the left fingers to lift. When reaching the highest point, push the front wheel bar to fold it.

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# **Using brakes**

Grasp both handles, press on the "red brake lock" to lock the rollator and it is well locked with a clear "click". Press on the "red brake lock" to unlock and it is unlocked with a clear "click".

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### **Using Tray**

Ensure the tray is installed with the rollator before using. If not installed, follow the steps to re-fit it.

Support the tray with both hands on the tray locks as per Figure 5 and lift them until the tray is in place, then gently press them down when the tray corners touch the columns on the bracket.

Ensure the component connects tightly on the supporting tube above the bag.

Ensure the tray corner touch the columns lightly to avoid damage of the columns. Otherwise it will destroy the function of the tray locks and lift the tray lock on the bracket to unlock them. Then take the tray out by lifting up flatly.

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### Maintenance

Ensure all parts of the rollator are in safe working status.

Check all screw to ensure all tubes are connected well and locked. Press and lock them if they are not. Check all the knobs and C-caps are completely inserted into the holes.

Change all damaged or worn parts timely.

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### Wheels checking

Clean the wheels with water or soft cleaner and dry them with clear cloth.

Drop some 3-in-1 lube if there is any strident noise.

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### Safety instructions

Use only as a walking device!
Use only on even and solid surfaces.

Walking device must not be used for sit on or as a wheelchair !It must not be used for transport of persons or animals!

The effect of the brakes can be affected by abrasion of the wheels.

Therefore test regularly the brakes.

Maximum load: 100 kg

Use tray only for small, light, but not for pointed or hot objects. Max. load: 5.0 kg

Use bag and basket only for transport of objects and do not overload. Max. load: each 5.0 kg

Do not expose the product to temperatures above 40°C for a long time, otherwise the steel pipes and the seat cover heat, which could lead to burns.

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# Cleaning/Maintenance

Clean regularly by hand with a damp cloth!
Do not use any aggressive detergents!
Screws of walking device must be checked regularly and if necessary retightened.

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### Reprocessing

The product is suitable for reprocessing, in line with manufacturer's recommendations.

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### Warranty

We issue a warranty of 24 months from date of purchase. Subject to technical alterations. Errors and omissions excepted.

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### **Technical Data**

**Dimensions** 

Length over all with basket 550 mm
Wdth over all 570 mm
Width between handgrips 460 mm
Handgrip hight from 814 to 941 mm

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### Type label

- 1. Product name
- 2. Article number
- 3. Date of manufacturing year/week
- 4. Serial number
- 5. Attention: See operating instructions

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# Scope of delivery contents

Walking device / Basket / Tray / Operating instructions.

Hersteller:

Drive Medical GmbH & Co. KG.

Leutkircher Straße 44 D-88316 Isny/Allgäu Germany

info@drivemedical.de Tel: +49 7562 9724-0 Fax: +49 7562 9724-25 **Drive Medical Ltd.** 

Ainleys Industrial Estate Elland, West Yorkshire, HX5 9JP www.drivemedical.co.uk

# **NOTIZEN / NOTES**

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**Your Specialist dealer:**